
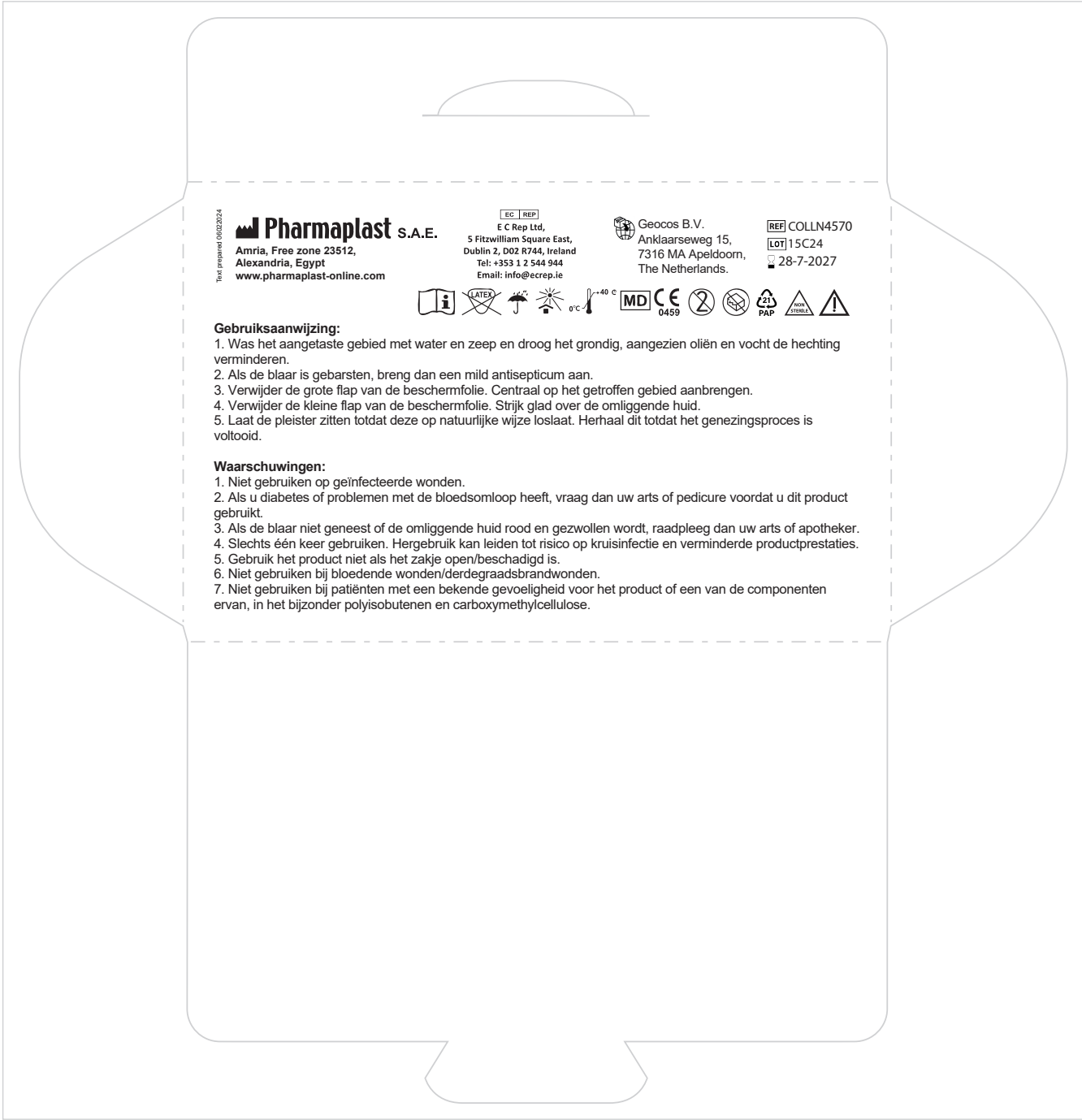
! Printing on the inside is possible at an additional cost.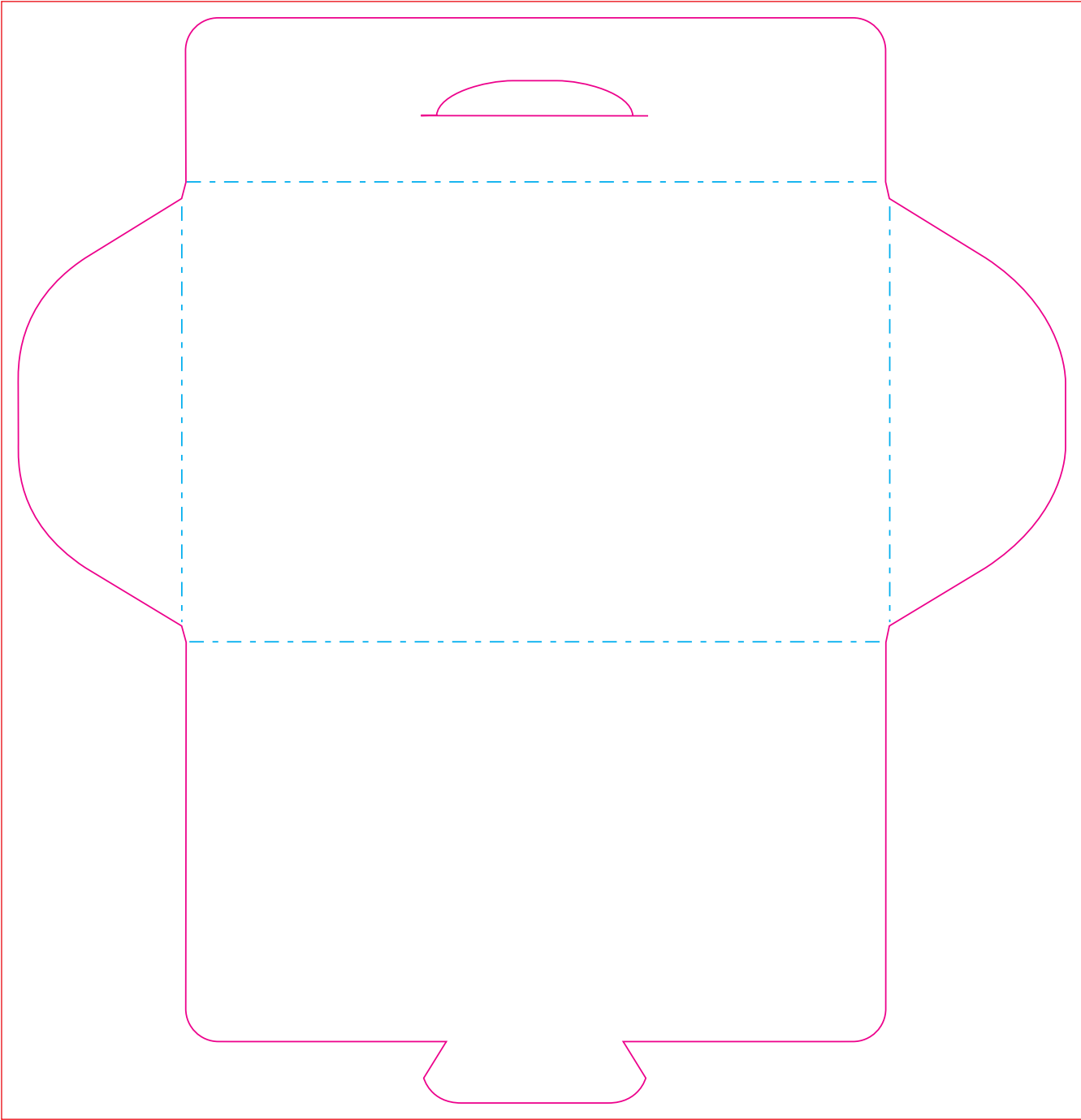

Bedrukkingsformaat 170 x 176 mm Printing size 170 x 176 mm Bedruckung Format 170 x 176 mm		Bedrukking full colour Printing full colour Bedruckung Euroskala		Positie bedrukking all-over Printing position all-over Bedruckungsposition Vollflächig	
Bedrukkings Formaat Printing Surface Bedruckbare Fläche	Vouwlijn Folding Line Falzlinie	Afloop Bleed Beschnitt	Geen Tekst Plaatsen No Text Here Kein Text Hier	Ingrediënten Ingredients Zutaten	

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Instructions for use:

1. Wash the affected area with soap and water and dry thoroughly since oils and moisture reduce adhesion.
2. If the blister has burst apply a mild antiseptic.
3. Peel off the large release liner flap. Apply centrally over the affected area.
4. Peel off the small release liner flap. Smooth over the surrounding skin.
5. Leave the plaster in place until it loosens naturally. Repeat until the healing process is complete.

Warnings:

1. Do not use on infected wounds.
2. If you have diabetes, or problems with circulation, ask your doctor or chiroprapist before using this product.
3. If the blister fails to heal or the surrounding skin becomes red and swollen, consult your doctor or pharmacist.
4. Use only once. Re-use could result in risk of cross-infection and reduced product performance.
5. Do not use the product in case the pouch is open/ damaged.
6. Do not use for bleeding wounds/ third degree burns.
7. Don't use on patient with known sensitivity to the product or any of its components specially Polyisobutenes and Carboxymethylcellulose.

 Pharmaplast S.A.E. Amria, Free zone 23512, Alexandria, Egypt www.pharmaplast-online.com	 E C Rep Ltd. 5 Fitzwilliam Square East, Dublin 2, D02 R744, Ireland Tel: +353 1 2 544 984 Email: info@ecrep.ie	 Geocos B.V. Ankaarsweg 15, 7316 MA Apeldoorn, The Netherlands.	REF COLLN4570 LOT 15C24 28-7-2027
	    		
	MD CE 0459    		
	    		

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